


Easy Reader™

Installation Guide and User Instructions

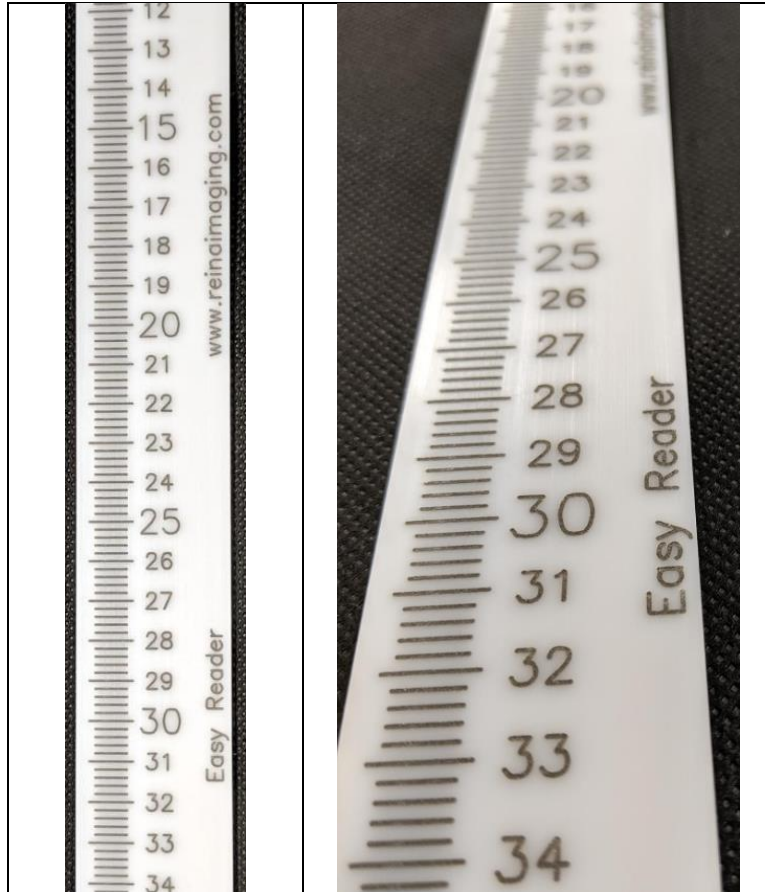

Models:

ER66SPH
ER34B
ER34BT
ER51B
ER51BT
ER34
ER51

Revision: June 2019

i. Intended Use

The Easy Reader™ is intended for use by the radiographic technologist as a visual aid for proper positioning of the patient during a radiographic examination and as a reference for combination of segment images (“stitching”).

ii. Safety and Compliance



General Cautions

- There are no general safety concerns regarding this device.

Compliance

The **Easy Reader™** is a Class 1 Medical Device (510k exempt) in the USA and is registered to:

Reina Imaging

6107 W. Lou Avenue
Crystal Lake, IL 60014 USA
Phone: 800-752-4918



The **Easy Reader™** has been labeled and declared:



This model has been classified as a Class 1, Annex VII device.

European Authorized Representative:

Pinchet Rx SRL
Contact: Paolo Pinchet
Via Don Gnocchi 24 20148 Milano Italy
Phone: 02 / 48707603

iii. Warranty

Reina Imaging warrants to the Original Purchaser that this product will be free from defects in its workmanship and materials, under normal use and service conditions, as described herein. “Defects” as used in this warranty, is defined as any imperfections which impair the use of the product. The company will repair or replace, at its option, without charge to the original purchaser only, the defective products or parts for a period of 1 (one) year from the date of purchase and while owned by the Original Purchaser. Should any component fail under normal use during this period, it is the buyer’s responsibility to contact customer service at 800-752-4918.

Not Covered Under this Warranty


Damage caused by user abuse, misuse, negligence, accidental misuse, failure to assemble properly and normal wear will not be covered under this warranty. Users are responsible for reading all instructions prior to using the product(s).

1. Installation

1.1. For models featuring a bracket:

ER66SPH	For Reina Imaging Stitching Platform with Handles (affixes to rear side of patient shield)
ER34B	For all Reina Imaging two panel devices except Trans-Former line of products
ER34BT	For all Reina Imaging two panel Trans-Former line of products
ER51B	For all Reina Imaging three panel devices except Trans-Former line of products
ER51BT	For all Reina Imaging three panel Trans-Former line of products

The Easy Reader™ bracket is designed to simply hang on the imaging device as shown in Figure 1.1 below:

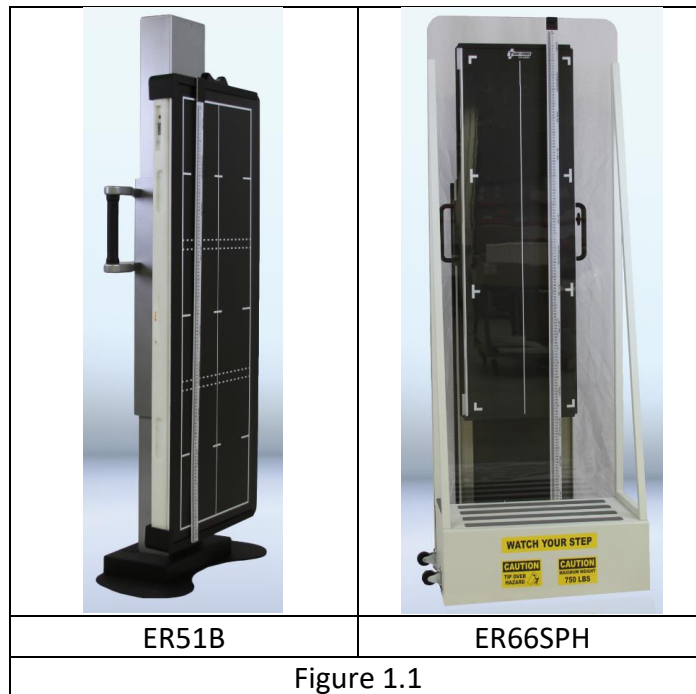

1.2. For tape-on models not featuring a bracket:

ER34	For all two panel devices
ER51	For all three panel devices

Use adhesive tape to affix the Easy Reader™ to your imaging device as desired.

2. Maintenance & Cleaning

2.1 Maintenance - The product does not require periodic maintenance.

2.2 Cleaning - Surfaces can be cleaned using standard disinfection products. Do not submerge components. Do not autoclave components.

ALL QUESTIONS AND CONCERNS SHOULD BE FORWARDED TO:

REINA IMAGING

6107 W. LOU AVE., CRYSTAL LAKE, IL 60014

800-752-4918

www.reinaimaging.com